



**BYD
CARE** 

PRODUCT INTRODUCTION

**BYD Single Use Face Mask
YY 0469-2011
EN14683-2014 Type IIR
ASTM F2100 Lv2**



Single-use Face Mask

SOFT & EASY TO BREATHE

Non-sterile



*Picture is for reference only, actual product may have differences.

BYD CARE

+ Use as **surgical mask**



Application

Worn by personnel to cover the mouth, nose and chin, the mask provides a physical barrier to block exhalation or ejection of pollutants from mouth and nose.



Product Introduction

BYD Single-use Face Mask consist of three layers of nonwoven material:



- Outer Layer**
Blue surgical use spunbond nonwoven
- Middle Layer**
Polypropylene melt-blown nonwoven with pathogen filtering
- Inner Layer**
White surgical use spunbond nonwoven

MATERIALS

This product consists of mask body, nose clip and ear loops. The mask is composed of three layers of filter material: The inner and outer covers are constructed from spunbond nonwoven fabric, for medical use. Polypropylene melt-blown nonwoven is layered between the inner and outer covers. The nose clip uses metal core plastic and the ear loops are made of nylon and spandex blended. Non-sterile product, one-time use only.

Wearing Instructions



1. Hold the mask by the ear loops with the colored side facing outwards and the nose bridge upwards, and place both ear loops around both ears.



2. Bend the nose clip to match the shape of the nose to prevent unfiltered air from entering.



3. Pull the mask to your chin to produce a tight seal.




Single-use Face Mask

SOFT & EASY TO BREATHE

Non-sterile

Product Specification

- 1 Mask length: > 170 mm(6.69in.)
- 2 Mask expansion width: 165 mm(6.49in.). Post-stack width is 95 mm(3.74in.) with three stacks in the middle. Each stack width is not less than 10 mm(0.39in.). The nose clip is located on the upper edge of the mask, and the outside of the mask is darker in color.
- 3 The width of the support part is not more than 10 mm(0.39in.), and the position of the upper nose clip is not more than 16 mm(0.63in.).
- 4 Nose clip: length > 80 mm(3.15in.); width approximately 3 mm(0.12in.)
- 5 Ear loop: 180 mm(7.09in.) in length and 3 mm(0.12in.) in diameter, composed of polyester, spandex and other materials, welded on the inner layer not more than 10 mm(0.39in.) from the edge.

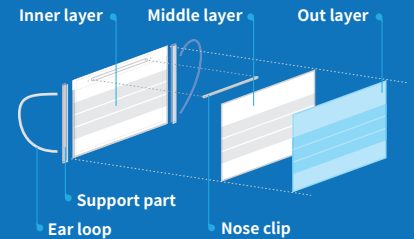

Parameter of BYD Single-use Face Mask

Name	Parameter
Product name	Single-use Face Mask
Material	Spunbond nonwoven fabric (for surgical use), polypropylene melt-blown nonwoven, metal core plastic nose clip, polyester and spandex ear loops
Model	Flat
Size	175 mm×95 mm / 6.89in.×3.72in.
Product application scope and purpose	For protection against inhalation of pathogens and airborne liquid droplets
Expiration date	2 years after production
Packaging specification	10pcs/bag; 50pcs/box; 2000pcs/case
Storage	Store in a well-ventilated place with relative humidity below 80%; avoid high temperatures and exposure to flame.

WARNING

- 1 The mask does not eliminate the risk of contracting any disease or infection.
- 2 Failure to properly use and maintain this product could result in illness or even death.
- 3 Only for one-time use, please destroy and dispose according to regulations after use.
- 4 Some materials in direct contact with the skin may cause allergic reactions or even anaphylaxis in sensitive individuals.
- 5 Please use within the validity period.
- 6 DO NOT use if package is damaged.
- 7 Please refer to the instructions before use.
- 8 For adult use only.
- 9 Do not wash.

All information presented in this document is based on the latest data available at the time of printing. BYD reserves the right to make changes at any time without prior notice.

Approval and Standard

BYD Single-use Face Mask are manufactured in accordance with industry recommended standards YY 0469-2011

Product model: FG2111 FDA product code QKR Australia TGA ARTG 332299

EN 14683 Type II



Single-use Face Masks

SOFT&EASY TO BREATHE



BYD CARES



SIZE & WEIGHT



98*220 mm

5 pcs / pack

NW 17.5 g

GW 18 g



220*100*122 mm

**10 pack (50pcs)
/ box**

NW 180 g

GW 230 g



520*460*518 mm

**40 box
(2000pcs) /**

case

NW 9.2 kg

GW 11.16 kg

**1 pallet=8case*2000pcs/case
(16000pcs)**

2layer*4case

1.2*1*1.161(H)m

NW 73.6 kg

GW 102.28 kg