

KN95 Particulate Respirator



Foldable (For single use only) **Non-sterile**

BYD CARE 

*Picture is for reference only,
actual product may have differences.

- ⊕ Use as a
surgical mask
- ⊕ Meets requirements
for a minimum **95%**
filtration efficiency
against solid and liquid



Application

It is intended for respiratory protection against non-oil particles and reduce user exposure to certain airborne particles including those from sanding, sawing, grinding, sweeping, bagging, or from processing minerals, coal, iron ore, flour, and certain other substances.



Standards

BYD KN95 mask complies with the national standard GB2626-2006 Self-Priming, Filter-Type Mask for Respiratory Protection. This standard specifies the technical requirements, testing methods and manufacturing procedures of the mask.

KN95

Particulate Respirator

Foldable (For single use only)



Non-sterile


Wearing instructions

- 1 Hold the ear loops in both hands with the metallic strip up.
- 2 Put the mask against your face covering both nose and mouth.
- 3 Place ear loops around both ears. Adjust mask over nose and under chin.
- 4 Adjust the flexible strip over bridge of nose.
- 5 Make sure the mask fits properly, covering the nose, mouth and chin.




Product composition

Design

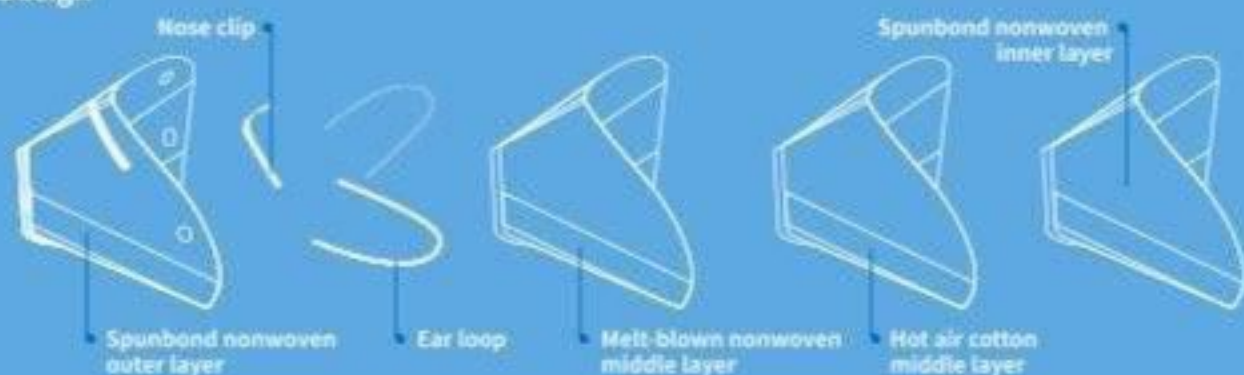

Material

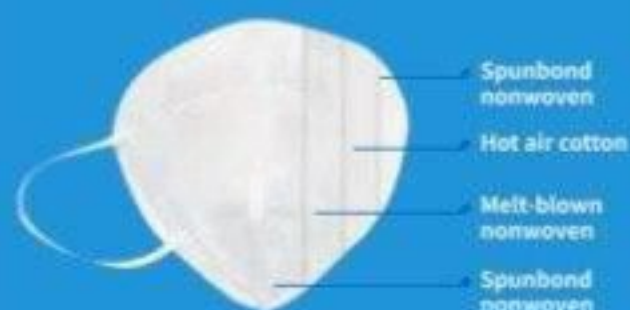

Specifications

Name	Parameter
Product name	KN95 Particulate Respirator
Model	Foldable
Size	110 mm × 160 mm/4.33in. × 6.30in.
Material	Spunbond nonwovens, melt-blown nonwovens, hot air cotton, nose clip, elastic ear loops
Storage	Store in a cool, dry, clean place away from fire. Store between -20°C and 38°C (-4°F and 100°F) with relative humidity below 80%.
Expiration date	3 years after production
Packaging specification	2pcs/bag; 50pcs/box; 1000pcs/case

WARNING

- The mask does not eliminate the risk of contracting any disease or infection.
- Improper use may lead to illness and even death.
- Some materials in direct contact with skin may cause allergic reactions or even anaphylaxis in sensitive individuals.
- For adult use only.
- Do not wash.

